


# Checking Your Blood Glucose Using TRUEresult twist

*A guide for testing your blood glucose*

## Simple Steps to Test Your Blood Glucose

### Meter, Test Strips & System Components

Front View

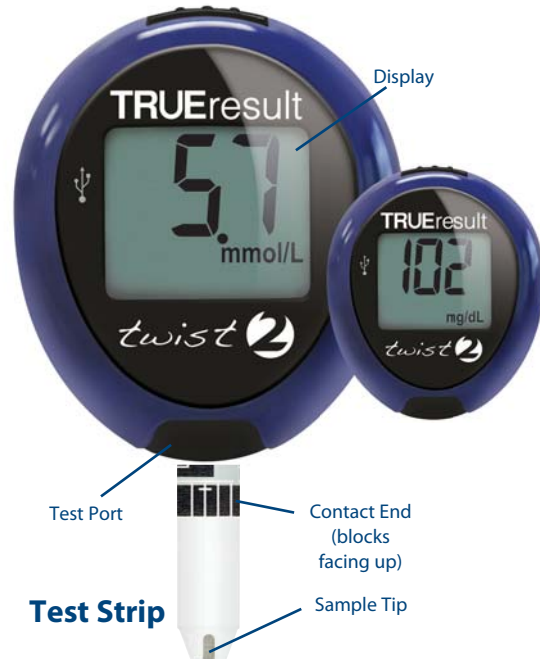

### Blood Glucose Testing

**1**



Wash your hands with warm, soapy water.

**2**



Remove Test Strip from vial and close vial immediately. Insert Test Strip into Test Port with Contact End (blocks facing up). The Meter turns on.

**3**



Lance your finger.

**4**



With Test Strip still in Meter, touch Sample Tip to top of blood drop and allow blood to be drawn into Test Strip. Remove from blood drop immediately after dashes appear across Meter Display.

**5**



In as fast as 4 seconds, glucose result will be displayed. Discard Test Strip in the appropriate container.



*Holding the Test Strip Sample Tip to the blood sample too long after the Meter begins testing may cause inaccurate results.*

This guide provides a brief overview on testing your blood glucose. If you need information about setting up your Meter with correct date and time, consult the "Getting Started" section of Owner's Booklet. For more detailed information on complete blood glucose testing procedure, consult "Testing Blood" section in the Owner's Booklet. *Note: Lancing device is for self-testing and intended for use by one patient only. Not suitable for use by healthcare or care workers.*