

Checking Your Blood Glucose Using TRUEresult twist

A guide for testing your blood glucose

Simple Steps to Test Your Blood Glucose

Meter, Test Strips & System Components



Blood Glucose Testing

1



Wash your hands with warm, soapy water.

2



Remove Test Strip from vial and close vial immediately. Insert Test Strip into Test Port with Contact End (blocks facing up). The Meter turns on.

3



Lance your finger.

4



With Test Strip still in Meter, touch Sample Tip to top of blood drop and allow blood to be drawn into Test Strip. Remove from blood drop immediately after the dashes appear across Meter Display.

5



In as fast as 4 seconds, glucose result will be displayed.

6



Hold Meter with Test Strip pointing down. Press Strip Release Button to discard Test Strip in the appropriate container.

This guide provides a brief overview on testing your blood glucose. For more detailed information on complete blood glucose testing procedure, consult "Testing Blood" section in the Owner's Booklet. Note: Lancing device is for self-testing and intended for use by one patient only. Not suitable for use by healthcare or care workers.